United Kingdom

Wellspect HealthCare
DENTSPLY IH Ltd, Building 3
The Heights, Weybridge
Surrey KT13 ONY
Tel: 0800 652 3350
Fax: 01453 791001
www.wellspect.co.uk

France

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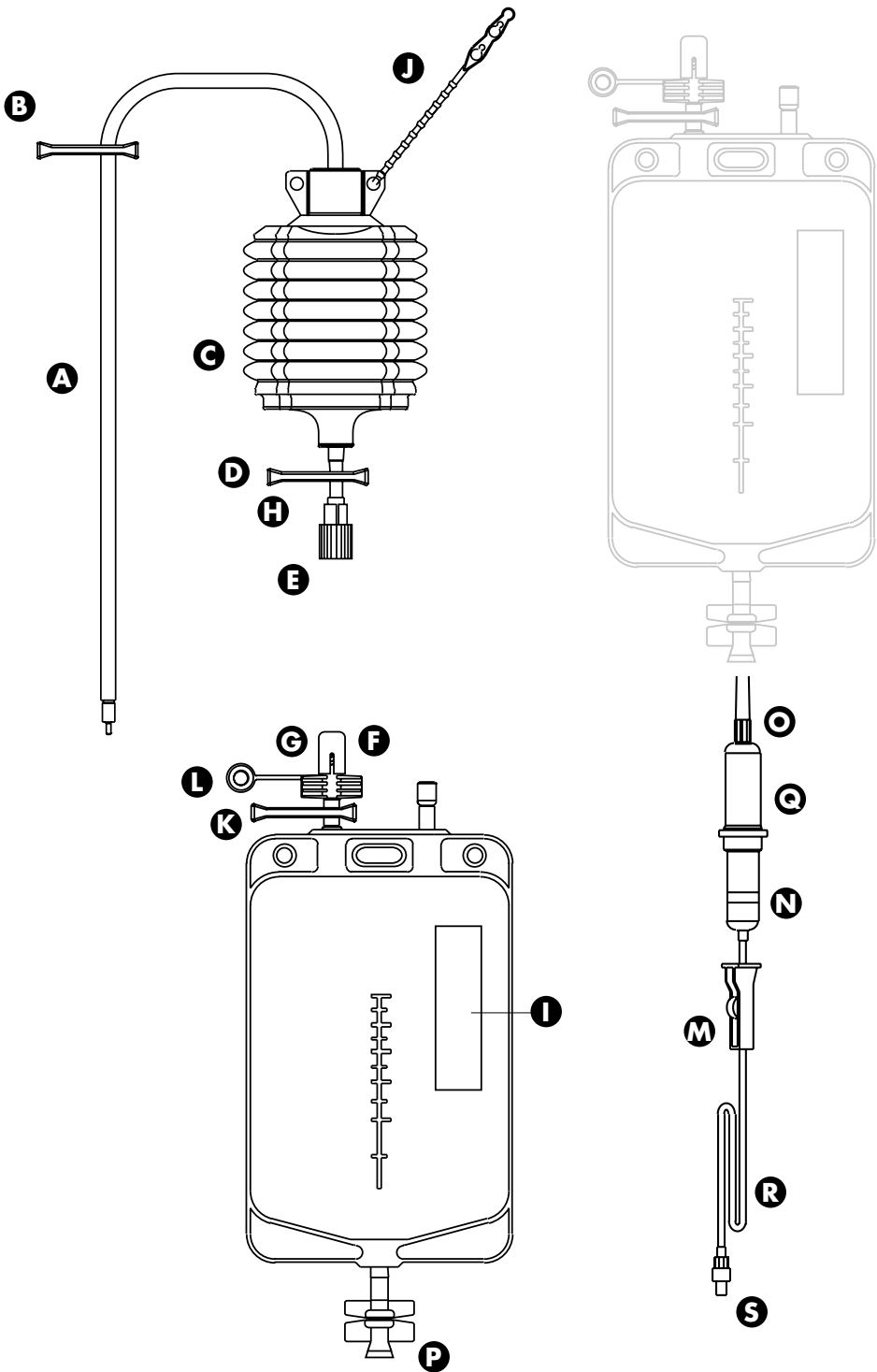
Product Description

- The Bellovac® ABT set contains a transfusion set, wound drainage system, catheter with trocar and an autotransfusion bag.
- The Bellovac ABT Set is a complete and easy-to-use active drainage and reinfusion system for replacement of blood lost following surgery.
- The Bellovac ABT has an initial negative pressure of 90 ± 10 mmHg/ 12 ± 1.3 kPa.
- The measuring scale of the Autotransfusion bag is approximate. An empty Bellovac ABT Autotransfusion bag weighs 45 g.
- The bellows can be filled to a maximum of 220 ml.
- Only Y-connector Sets with Ref no 68376, 68377, 68378, 68385, 68386 and 68387 should be used with Bellovac ABT.
- Only Transfusions Sets with Ref no 68381 and 68379 should be used with Bellovac ABT.

Free from DEHP, pyrogens and latex.

Radiodetectable catheter.

Indication

The Bellovac® ABT set is used in adults for postoperative drainage, collection, filtration and reinfusion of blood, in knee and hip surgery.

Contraindications

- Malignancy, infection or contamination in the area of the operative site.

Warning and precautions

- Defective red blood cells, e.g. sickle cell anemia.
- In case of fever or chills, reinfusion should be discontinued since it indicates high levels of inflammatory mediators.
- Do not leave the drainage in situ longer than necessary. Delayed withdrawal may cause discomfort and tissue damage.
- Reinfusion only via peripheral veins.

Handling Cautions

- To avoid air embolism, do not apply excessive pressure to the Autotransfusion bag.
- To avoid air embolism, remember to close the roller clamp (M) before changing the Autotransfusion bag or the Transfusion Set.
- To avoid air embolism, avoid running the transfusion line empty during reinfusion.
- Bellovac ABT is a single-use product. If reused Wellspect HealthCare cannot guarantee the functionality nor the safety of the product. Reuse may lead to infection.

Instructions for use

Set up

1. Open the Bellovac ABT Set package using aseptic procedures.
2. Place the catheter in situ. Remove the trocar, secure the catheter, and connect the catheter to the inlet tubing (A).
3. To activate the system and evacuate air from the operation area, close the inlet clamp (B), compress the bellows (C), and open the inlet clamp again. Repeat the procedure until blood starts to flow, or the bellows remain in a compressed, vacuumed state.
4. Close the outlet clamp (D).
5. Remove the protective cap (F) from the bag inlet (G).
6. Attach the Autotransfusion bag to the outlet port (H) of the bellows. Mark (I) with the identity of the patient, what time the collection started, and definite time when the collection should be terminated (blood collection should be completed within 6 hours of the start of collection) on the bag.
7. Hang the Bellovac ABT in an upright position, using the bed rail strap (J).
8. Send the Transfusion Set along with the patient to the postoperative ward.

Notes

- If the Extension Line is to be used (e.g. for Jehovah's Witnesses), connect it between the Autotransfusion bag and the Bellovac ABT outlet port (H) before starting collection.
- If the Transfusion Set is to be connected to the venipuncture site before it is filled with blood, it has to be primed with saline solution.

Blood Collection

1. Make sure that the Autotransfusion bag (I) is correctly marked with patient identity and what time the collection started.
Note: Blood collection should be completed within 6 hours of the start of collection.
2. To empty the contents of the bellows into the Autotransfusion bag, close the inlet clamp (B) of the bellows, and open the outlet clamp (D).
3. Compress the bellows (C) slowly using the palms of both hands, transferring the liquid into the bag.
4. Close the outlet clamp (D), and open the inlet clamp (B) to continue drainage.

Bag exchange

1. Make sure that the outlet clamp (D) of the bellows is closed.
2. Close the inlet clamp (K) of the bag, disconnect the full Autotransfusion bag, and seal off with the cap (L).
3. Connect a new bag by turning the screw connectors tightly. Be careful not to contaminate the connectors.
The same procedure is applied when changing to a Bellovac® Exchange drainage bag.

Reinfusion

1. Open the Transfusion Set using aseptic procedures.
2. Position the roller clamp (M) on Transfusion Set adjacent to the drip chamber (N) and close off. Remove the protective caps from the piercing spike (O) and the spike port (P) of the Autotransfusion bag. Insert the piercing spike (O) into the spike port (P).
3. To expel any air from the Autotransfusion bag and to prime the filter (Q) and drip chamber (N), invert the Transfusion Set and Autotransfusion bag, open the roller clamp (M), and gently squeeze the Autotransfusion bag to expel any air through the Transfusion Set and to retrograde prime the filter and drip chamber. Fill up to the mark on the drip chamber.
Close the roller clamp ensuring that the level is maintained in the drip chamber.
4. Suspend the Autotransfusion bag on the drip stand.
5. Carefully open the roller clamp (M) and expel all the air from the tubing (R). Close the roller clamp.
6. Remove the protective cap (S) from the end of the Transfusion Set tubing and attach to venipuncture device.
7. Start reinfusion.
 - Do not use the Transfusion Set for more than one unit of autologous blood. A new Autotransfusion bag and a new Transfusion Set should be used for each collected unit.
 - If the flow through the microfilter (Q) is impaired due to clots, the Transfusion Set should be exchanged. If the flow through the replacement Transfusion Set is impaired, the transfusion must be discontinued.

Continuation of drainage

After six hours of blood collection, the Autotransfusion bag has to be exchanged for a standard Bellovac® Exchange drainage bag. The Bellovac ABT Set can now be used as an ordinary wound drainage system.

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GB**Bellovac ABT product range****Bellovac® ABT Set**

Ref no 68396	Bellovac® ABT Set FG 10 with trocar
Ref no 68397	Bellovac® ABT Set FG 14 with trocar
Ref no 68398	Bellovac® ABT Set FG 18 with trocar

Bellovac® ABT Accessories

Ref no 68376	Y-connector and catheter FG 10
Ref no 68385	Y-connector and catheter FG 10 with trocar
Ref no 68377	Y-connector and catheter FG 14
Ref no 68386	Y-connector and catheter FG 14 with trocar
Ref no 68378	Y-connector and catheter FG 18
Ref no 68387	Y-connector and catheter FG 18 with trocar
Ref no 68379	Companion Set (Autotransfusion bag and Transfusion Set)
Ref no 68380	Extension Line
Ref no 68381	Transfusion Set
Ref no 68337	Exchange drainage bag – PVC-free

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