

Peel-open tab.

Adhesive tab on back for reliable hanging.

Instant activation - sterile salt solution included, squeeze and it's ready to use.

Adjustable Insertion Grip for better grip and control with no need to touch the catheter tube.

Not manufactured with PVC, phthalates or latex.



LoFric catheters are coated with the unique Urotonic™ Surface Technology. This ensures the catheter remains smooth and safe, both during insertion and withdrawal.

Smooth eyelets.



Foldable into handy, discreet pocket size.



User friendly customer box.

Intended use

LoFric® Origo™ single use urinary catheter is intended for intermittent urinary catheterization.

Sterilization and shelf life

Sterilized by irradiation. Use before expiry date on package.

Packaging

30 catheters per customer box. 4 customer boxes per shipping box.

Material composition

Hydrophilic surface coating: Polyvinyl pyrrolidone (PVP).

Catheter: Polyolefin-based elastomer (POBE).


Sachet: Laminate foil of polyethylene terephthalate (PET), aluminum and polyethylene (PE). Sterile water with sodium chloride (NaCl).

Item package: Top foil: Laminate foil of polyethylene terephthalate (PET) and polyethylene (PE).

Bottom foil: Polyolefin.

Product range

Reference	Type	Length	Size
442 08 00	LoFric® Origo™ Nelaton (paediatric)	30 cm	CH 08
442 10 00	LoFric® Origo™ Nelaton (paediatric)	30 cm	CH 10
442 12 00	LoFric® Origo™ Nelaton (paediatric)	30 cm	CH 12
443 10 00	LoFric® Origo™ Nelaton (male)	40 cm	CH 10
443 12 00	LoFric® Origo™ Nelaton (male)	40 cm	CH 12
443 14 00	LoFric® Origo™ Nelaton (male)	40 cm	CH 14
443 16 00	LoFric® Origo™ Nelaton (male)	40 cm	CH 16
443 18 00	LoFric® Origo™ Nelaton (male)	40 cm	CH 18
444 10 00	LoFric® Origo™ Tiemann (male)	40 cm	CH 10
444 12 00	LoFric® Origo™ Tiemann (male)	40 cm	CH 12
444 14 00	LoFric® Origo™ Tiemann (male)	40 cm	CH 14
444 16 00	LoFric® Origo™ Tiemann (male)	40 cm	CH 16
444 18 00	LoFric® Origo™ Tiemann (male)	40 cm	CH 18

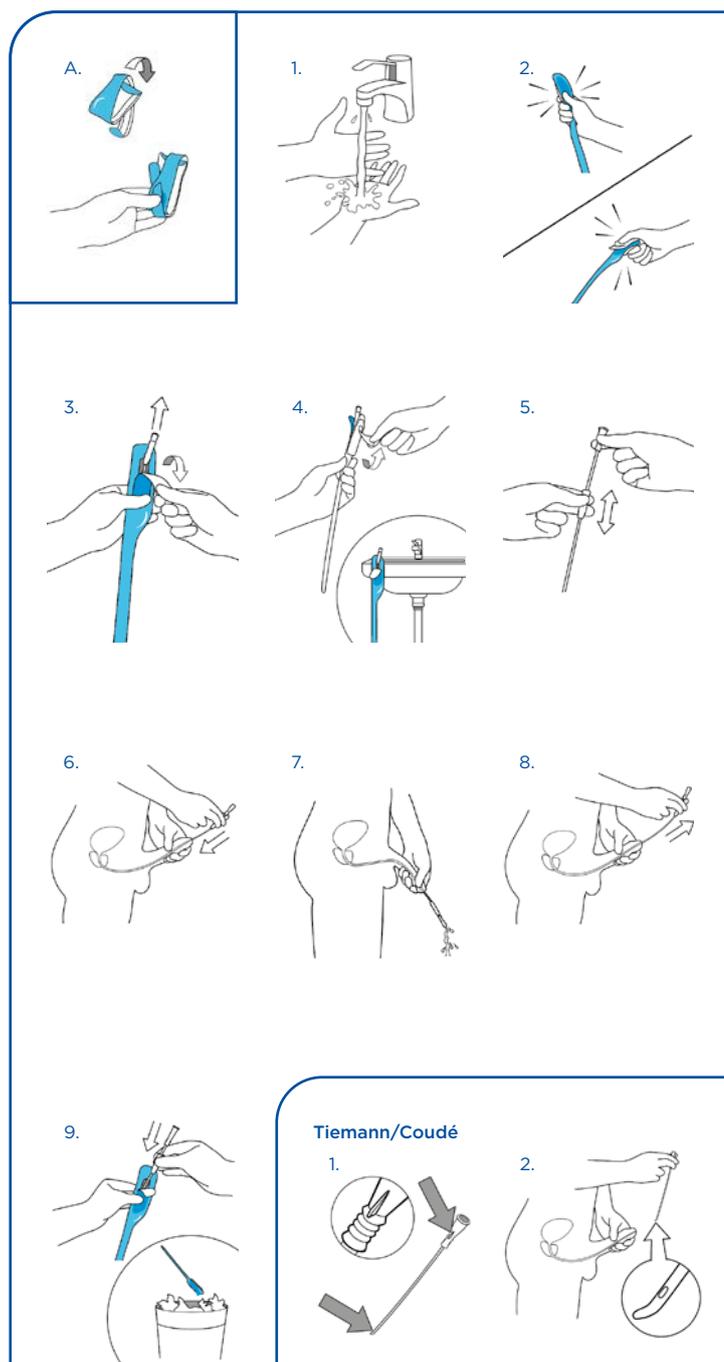

Ensure proper training of the catheterization procedure is provided to the user.

LoFric catheters are for prescription use only.

A. Tips for storing before use.

Foldable to pocket size.

Use the slit to keep the catheter folded.

INSTRUCTIONS FOR USE

1. Wash your hands thoroughly with soap and water.
2. Press to release the salt solution and the catheter is ready to use.
3. Pull the tab down to open.
4. Optional: Use the adhesive tab on the reverse side to attach the product to a dry, clean surface.
5. Take out the catheter.
Optional: Pull and adjust the Insertion Grip located on the funnel, to control insertion without having to touch the catheter tube.
6. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
7. Angle the penis down as urine begins to flow through the catheter.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

Special instruction for Tiemann/Coudé catheter, with slightly curved tip.

A special technique is required when using a Tiemann/Coudé. Speak to your healthcare professional for training and advice.

1. Note where the marker on the funnel is in relation to the curved catheter tip before inserting the catheter. It will guide you keeping the curved tip in the right direction during use.
2. Keep the curved tip upwards towards the stomach during insertion and throughout catheterization, including withdrawal. Or follow specific instructions given by your healthcare professional.

IMPORTANT INFORMATION

Use only the salt solution included to wet the catheter.

Do not place the catheter on paper or cloth as these materials can absorb the wet coating.