

Navina Mini

Description

Navina Mini is intended for transanal irrigation by instilling a low volume of water into rectum through a single use rectal cone.

Indications for use

Navina Mini is indicated for adults, adolescents and children from 3 years who suffer from incomplete emptying and/or fecal incontinence.

Contraindications

Do NOT use Navina Mini if you have one or more of the following:

- Known anal stenosis
- Active inflammatory bowel disease
- Acute diverticulitis and diverticular abscess
- Colorectal cancer
- Ischemic colitis
- You are within three months of anal or colorectal surgery

Since the list may not be exhaustive, healthcare professionals should always consider individual user factors as well.




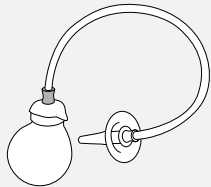

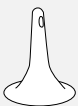
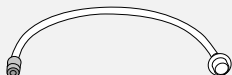
Durable design

- Water container 60 uses
- Extension tube 60 uses
- Rectal cone single use

Material composition

The main components of Navina Mini (water container, rectal cone and extension tube) are made of thermoplastic elastomer (TPE).

Navina Mini are not manufactured with latex, PVC or phthalates.


				
Navina Mini assembled	Navina Mini assembled with extension tube	Water container 185 ml 60 uses	Rectal cone Single use	Extension tube (optional) 60 uses

Reference	Type	Specification
6931000	Navina Mini Start Set	1 water container, 5 rectal cones, 1 extension tube
6930700	Navina Mini Set	1 water container, 30 rectal cones
6930800	Navina Mini Extension Set	1 water container, 30 rectal cones, 1 extension tube
6930900	Navina Mini Refill Set	30 rectal cones


Navina Bowel Care. A complete range of bowel care products



Navina Irrigation System, the only irrigation system with electronic or manual control, for constipation and/or bowel leakage.

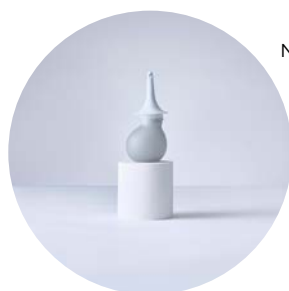

Navina Mini, for incomplete emptying and/or soiling.

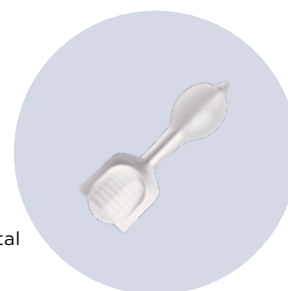

Navina Insert, engineered to prevent accidental bowel leakage.

At Wellspect we develop innovative continence care solutions that improve quality of life for people with bladder and bowel problems. We inspire our users to build self-confidence and independence as well as good health and well-being. We have been leading the industry for over 40 years with our product brands LoFric® and Navina™. We always aim to minimize the environmental impact of our products and passionately strive to become climate neutral. We work together with users and healthcare professionals to improve clinical outcome in a sustainable way, now and for the future.

Wellspect. A Real Difference.

For more information about our products and services, please visit [Wellspect.com](https://www.wellspect.com).

Join the conversation on LinkedIn, Twitter, Facebook and Instagram.

[wellspect.com](https://www.wellspect.com)

Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.

Phone: +46 31 376 40 00. www.wellspect.com

Navina Mini



Manufacturer: Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.
Phone: +46 31 376 40 00. www.wellspect.com

